

ThinPrep® Pap Test Rovers® Cervex-Brush® collections

Frequently Asked Questions

What do I do if the vial is knocked over?

If the vial is knocked over after the sample has been taken, look to see if any fluid still remains in the vial. If about one third of the fluid still remains in the vial, then the vial can still be sent to the lab for processing. Make a note on the request form to inform the laboratory of the sample spillage. Do not 'top up' the vial from a new vial as the lab will make any necessary level adjustments.

Can I take a ThinPrep® sample if the patient is menstruating?

No. Peri-menstrual samples should be avoided.

Can I use the Rovers® Cervex-Brush® where there is stenosis of the os?

It is still possible to use the Rovers® Cervex-Brush® however an endocervical brush may also be used at the discretion of the practitioner. Both sampling devices should be rinsed in the same patient vial.

Can I use the Rovers® Cervex-Brush® for vault samples?

Yes. You can use the Rovers® Cervex-Brush® to take vault samples. The device should be applied by using a sweeping action.

Can I use the Rovers® Cervex-Brush® if there is an extensive ectropion area?

Yes. You can use the Rovers® Cervex-Brush® when there is an extensive ectropion area. Collect the sample using a circular sweeping action.

Can I use a lubricant?

If lubrication of the speculum is required, a little warm water should be used. If necessary a small amount of water-soluble carbomer-free lubricant should be applied sparingly to the outer portion of the speculum with great care to avoid the tip as it may interfere with the ThinPrep® Pap Test process.

Do I need to take a sample for HPV testing?

No. HPV testing can be performed on the same material collected for the ThinPrep® Pap Test.

Do's and Don'ts

Do

- Do use either the Rovers® Cervex-Brush® (broom-like device) on its own or a plastic spatula and endocervical brush combination.
- Do rotate the Rovers® Cervex-Brush® 5 times in a clockwise direction through 360°.
- Do insert the endocervical brush such that you can see the lower row of the bristles and make a quarter to one half turn.
- Do insert the Rovers® Cervex-Brush® device immediately into the vial by pushing it into the bottom of the vial 10 times, forcing the bristles apart.
- Do as a final step, swirl the brush vigorously to further release material. Visually inspect the Rovers® Cervex-Brush® device to ensure that no material remains attached. Discard the collection device.
- Do replace the lid and tighten so that the black torque line on the cap passes the black torque line on the vial to avoid leakage.
- Do keep the unlabelled portion of the sample vial free of labels so that the contents can be seen.
- Do send samples to the laboratory without delay.

Don't

- Do not routinely clean the cervix or take a cervical swab before taking a cervical sample.
- Do not use non water-soluble lubricants containing carbomers.
- Do not leave the collection device sitting in the vial whilst dealing with the patient.
- Do not over tighten the lid onto the vial.
- Do not detach the head of the Rovers® Cervex-Brush® into the vial.
- Do not place multiple labels on the outside of the vial.
- Do not apply any practice bar-coded labels vertically on the vial, apply them horizontally.
- Do not apply any labels over the lid of the container.
- Do not use expired vials.
- Do not delay the sending of the vials to the laboratory.



ThinPrep®
PAP TEST

References

1. Cervicovaginal Cytology Based on the Papanicolaou Technique; Approved Guideline - Third Edition (Clinical and Laboratory Standards Institute GP15-A3).
2. Hologic internal study, data on file.
3. Rovers® Cervex-Brush® Combi, Information For Use, 2010-11.

Specifications are subject to change without prior notice. Hologic, Science of Sure, PreservCyt, ThinPrep and associated logos are trademarks and/or registered trademarks of Hologic, Inc. and/or its subsidiaries in the United States and/or other countries. All other trademarks, registered trademarks, and product names are the property of their respective owners. This information is intended for medical professionals and is not intended as a product solicitation or promotion where such activities are prohibited. Because Hologic materials are distributed through websites, eBroadcasts and tradeshows, it is not always possible to control where such materials appear. For specific information on what products are available for sale in a particular country, please contact your Hologic representative or write to australia@hologic.com

This is a general information tool for medical professionals and is not a complete representation of the product(s)' Instruction for Use (IFU) or Package Insert, and it is the medical professionals' responsibility to read and follow the IFU or Package Insert. The information provided may suggest a particular technique or protocol however it is the sole responsibility of the medical professional to determine which technique or protocol is appropriate. At all times, clinicians remain responsible for utilizing sound patient evaluation and selection practices, and for complying with all applicable rules and regulations regarding accreditation, anesthesia, reimbursement, and all other aspects of in-office procedures. In no event shall Hologic be liable for damages of any kind resulting from your use of the information presented.